CHROMagar™ MRSA

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BIBLIOGRAPHY

Tandé D. *et al.* 2003. Interest of use in routine of CHROMagar MRSA medium for detection of Methicillin Resistant *Staphylococcus aureus* carriers in intensive care unit. Poster presented during RICA1 2003 meeting, Paris, France.

Taguchi *et al.* 2004. Evaluation of a New chromogenic medium for isolation of MRSA. The Journal of the Japanese Association for Infectious Diseases. January 2004, p 54-58.

De Giallully *et al.* 2004. Sensitivity and Specificity of the new Chromogenic Media CHROMagar™ MRSA for Detecting methicillin Resistant Staphylococcus aureus. Poster 04-GM-A-1729 presented during 2004 ASM meeting.

Diederer *et al.* 2005. Performance of CHROMagar MRSA Medium for Detection of Methicillin-Resistant *Staphylococcus aureus*.. J. Clinical Microbiology. **43**: 1925-1927.

Available from CHROMagar :

CHROMagar™ Candida
Differentiation of major pathogenic *Candida* species

CHROMagar™ Orientation
Differentiation of urinary tract pathogens

Rambach™ Agar
Detection of *Salmonella* spp

CHROMagar™ Salmonella
Detection of *Salmonella* including *S. Typhi*

CHROMagar™ Salmonella Plus
Detection of *Salmonella* according to the ISO 6579:2002 norm

CHROMagar™ O157
Detection of *E.coli* O157

CHROMagar™ E.coli
Detection and enumeration of *E.coli*

CHROMagar™ ECC
Detection and enumeration of *E.coli* and coliforms

CHROMagar™ Liquid ECC
Broth for pad technique for *E.coli*-coliforms

CHROMagar™ Staph aureus
Detection and enumeration of *Staphylococcus aureus*

CHROMagar™ MRSA
Detection of MRSA including low level MRSA

CHROMagar™ Listeria
Detection and enumeration of *Listeria monocytogenes*

CHROMagar™ Vibrio
Detection and enumeration of *Vibrio parahaemolyticus*, *Vibrio vulnificus* and *Vibrio cholerae*

CHROMagar™ VRE
Detection of *E.faecium* VRE & *E.faecalis* VRE

CHROMagar™ StrepB
Detection of Group B *Streptococci*

CHROMagar™ and Rambach™ are trademarks of Dr. A. Rambach

Visit CHROMagar on internet via <http://www.chromagar.com>

CHROMagar
Microbiology

4, place du 18 Juin 1940
75006 Paris France
Fax: (33-1) 45 48 06 06

NT-EXT-012 Version 5